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zu 15

zu 16


# Food Intolerance Test Instructions

## Dear Patient,

To provide for high diagnostic precision of the test please comply with the instructions for sample withdrawal.

## The Blood Withdrawal Set consists of

- **1 Request Form** with Test Instructions
- **2 lancets** (1 as reserve)
- **1 disinfection wipe**
- **1 band-aid**
- **1 capillary blood tube**
- **1 transport tube**
- **1 label for the name**

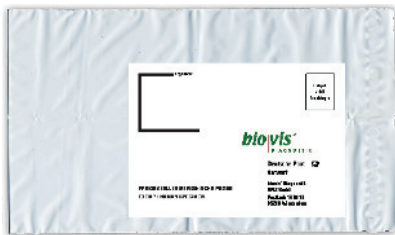Please only take samples from Monday to Wednesday and ship at the day of sample withdrawal.

Please include the completed patient form. Do not forget to note the date of withdrawal. Without this information we cannot process your analysis. If you have questions, please call us.

Please carefully read the instructions before you start. Please do not take samples after extraordinary strain or exercise.

If you have problems withdrawing the blood or something is missing in the kit, please contact our customer service.

## Warning

Keep out of the reach of children.

Lancets are only for personal use.

Please be careful with blood specimen.

**Please do not carry out the test if you...**

...suffer from blood transmitted diseases (e.g. hepatitis or HIV)

...are pregnant or breast-feeding.

**biovis'**  
DIAGNOSTIK

1. Open the box and check if the utensils are all contained.
2. Please read instructions carefully and take sample accordingly.
3. Completely fill in the patient form.
4. Put the blood tube on an even surface. Make sure it cannot tip.
5. Hold your hand under warm water for one minute.
6. Dry your hand carefully
7. Take the disinfection wipe out of the foil and disinfect the tip of one finger.
8. Please do not use lancets without or with loose protective cover. Hold lancet like shown on the picture. Turn the yellow protective cover once and carefully remove it.
9. Put your hand on solid surface, hold lancet between index and middle finger and apply slantwise on the tip of the finger. Now press your thumb hard on the yellow button.
10. Hold your hand horizontally at your hip and massage your finger in direction of the puncture until blood drops develop. **Please discard the first drop of blood. It contains particularly high amounts of tissue fluid.**
11. Hold your finger diagonally at the capillary of the blood vial (standing vertically) so that the blood can drop into it. Keep on massing your finger to make sure there is enough blood.
12. Only finish sample withdrawal when the mark (200  $\mu$ l) is reached. Tap the tube carefully on the table several times to make sure that the blood out of the capillary can drain. Remove the lid with capillary.
13. Remove the cover at the bottom of the tube and close the tube by pressing the cap until it locks in place.
14. Write your name (ID) and the date of blood withdrawal on the label.
15. Remove the foil on the back of the label and stick it on the filled blood tube.
16. Now put the filled blood tube into the larger transport tube.
17. Put tube and request form into the shipping bag and send this immediately to biovis.

Step 4

Step 5

Step 6

Step 7

Step 8

Step 9

Step 10

Step 11

Step 12

Step 13

Step 14

Step 15

Step 16