

Östrogen-Metaboliten (F325)



Testanleitung

Im Testset sind folgende Materialien enthalten:

- 1 x Anforderungsbogen**
- 1 x Testanleitung**
- 1 x Zentrifugenröhrchen stabilisiert**
- 1 x Transferpipette**
- 1 x Transportumröhrchen**
- 1 x Namensetikett**

Anleitung:

Das Urinröhrchen mit weißer Kappe muss mit dem zweiten Morgenurin bis zur Markierung oben gefüllt werden.

- Fangen Sie den zweiten Morgenurin nach dem Aufstehen in dem Urinauffangbecher auf (oder in einem entsprechend sauberen Gefäß).
- Befüllen Sie mit der beiliegenden Pipette das Urinröhrchen mit der weißen Kappe bis zur Markierung oben - ca. 5 mm unter dem geschlossenen Deckel. Bitte achten Sie darauf, dass die Stabilisatorsubstanz in dem Röhrchen verbleibt.
- Plastikbecher und Pipette können anschließend entsorgt werden.

Bitte Probengefäße unbedingt beschriften mit:

- **Namen**
- **Datum**
- **Material = Urin ankreuzen**

Bitte geben Sie die beschrifteten Urinmonovetten in das Transportumröhrchen (Schutzgefäße für den Postversand) und diese zusammen mit dem ausgefüllten Anforderungsbogen in die Postversandtasche.



Wichtiger Hinweis: Bringen Sie das Testset möglichst schnell auf den Postweg.

Ein Versand über das Wochenende und über Feiertage hinweg sollte vermieden werden.

Estrogen Metabolites (F325)



Test Instructions

The test set contains the following materials:

- 1 x Request Form**
- 1 x Test Instructions**
- 1 x Centrifuge Tube stabilized**
- 1 x Transfer Pipette**
- 1 x Transport Tube**
- 1 x Name Label**

Test Instruction:

The urine tube with white cap must be filled up to the mark above with the second morning urine.

- After getting up, catch the second morning urine in the urine collection cup.(or in an appropriately clean container).
- Use the enclosed pipette to fill the urine tube with the white cap up to the mark above - approx. 5 mm below the closed lid. Please ensure that the stabilizer substance remains in the tube.
- Plastic cup and Pipette can then be disposed.

Please be sure to label sample containers with:

- **Name**
- **Date**
- **Material = tick urine**

Now put the labelled saliva specimen tubes into the transport tube and put it into the shipping envelope together with completed request form. Ship as soon as possible!



Important: Send the test set by post as quickly as possible.

Shipment over the weekend and public holidays should be avoided.